BENNY / BENNY XL

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Table of contents

Preface.....	7
Declaration of conformity.....	7
Service life.....	7
Disposal.....	7
Identification plate.....	7
Reuse.....	7
Warranty.....	7
Parts of the product.....	8
Intended purpose.....	8
Indication.....	8
Scope of delivery.....	8
Safety instructions.....	8
Assembly.....	9
In the bathtub.....	9
Cleaning Maintenance.....	9
Technical data.....	9

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year/month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available at www.drivedevilbiss.de or from your specialist dealer.

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.


Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): Rubber feet

Parts of the product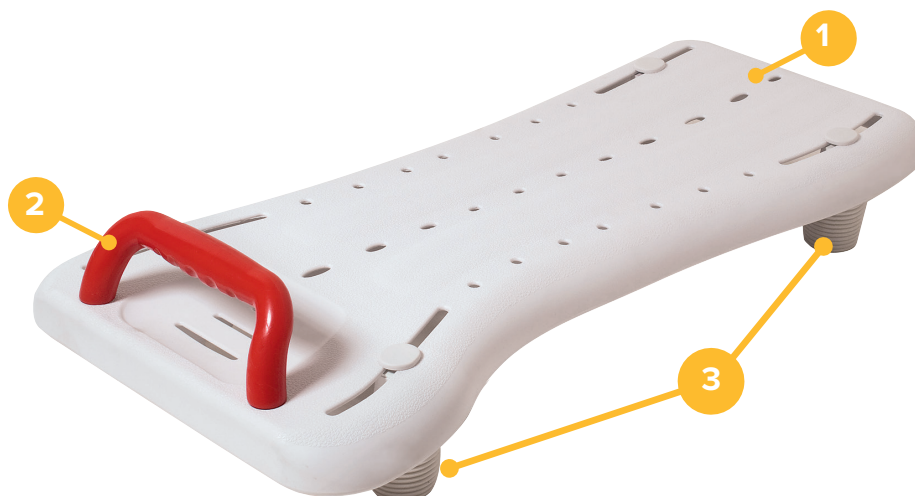

BENNY / BENNY XL

1. **Bath board**
2. **Handle**
3. **Stop bolts**

Intended purpose

Bath boards are placed on the edge of the bath. This reduces the distance you need to raise/lower yourself using your own strength. They are used when there are significant functional limitations of the upper and/or lower extremities. With the aid, bathing is a largely independent task again. However, using this aid presupposes that there is sufficient residual functionality in terms of torso position and use of the extremities to be able to change position.


Indication

INDICATION:

Impossibility to be securely seated or sit down in the bathtub due to impaired function of the lower extremity with limited or absent mobility and muscle strength with simultaneously preserved stable torso position and sufficient residual functions of the upper extremities.

Scope of delivery

1 x A	Seat unit
4 x B	Stop bolts
1 x C	Handle
1 x	Instructions for use

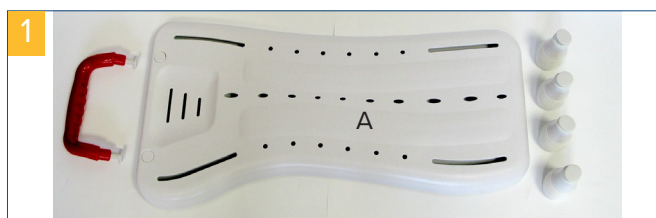Safety instructions


- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Proceed slowly and carefully when sitting down. Do not drop!
- Caution – risk of slipping/falling!

- Do not stand on the bath board.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.

Assembly


Handle

If necessary, remove both plastic screws from the handle. Insert these from below into the openings provided in the bath board. The white cover parts for the holes are then pushed upwards and fall out. Insert handle from above – facing inwards (**pic. 2**) – into the openings provided on the bath board and tighten the fixing screws underneath (such as with a coin) to secure the handle (**pic. 3**).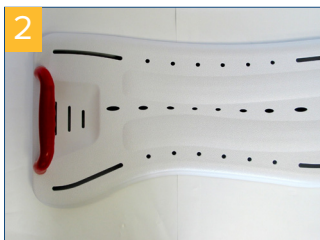

Stop bolts

If necessary, remove the plastic screws on top from the stop bolts. Insert a screw from above downwards into each slot on the bath board, position rubber rings against the screw from underneath (**pic. 4+5**) and gently screw on the stop bolts from underneath (**pic. 6**).

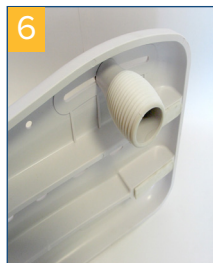Do not tighten the stop bolts yet so that they can be adjusted within the slot to the width of the tub afterwards.

In the bathtub

Place bath board across the tub on the edge of the tub. The handle must face the wall. The bath board must be positioned properly on the right and left. Push the stop bolts in each slot all the way outwards to the edge of the tub so that the board is secured in place.

Now tighten the screws (**pic. 7**).

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60° C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

Technical data

Product	Benny 68 cm	Benny 73 cm
Article number	550100100	550100200
HMV (medical aids register) no	04.40.02.0017	04.40.02.0016

Technical data		
Overall width	35 cm	35 cm
Overall length	68 cm	73 cm
Adjustment range (interior width of tub)	40 - 63 cm	43 - 68 cm
Total weight	2.1 kg	2.6 kg
Max. load	150 kg	150 kg

Product	Benny XL 68 cm	Benny XL 73 cm
Article number	550100300	550100400

Technical data		
Overall width	35 cm	35 cm
Overall length	68 cm	73 cm
Adjustment range (interior width of tub)	40 - 63 cm	43 - 68 cm
Total weight	2.2 kg	2.7 kg
Max. load	225 kg	210 kg

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Drive Medical GmbH & Co. KG