**drive**

**DeVilbiss**  
HEALTHCARE

Gebrauchsanweisung  
Operating manual  
Mode d'emploi  
Gebruiksaanwijzig

# Solido

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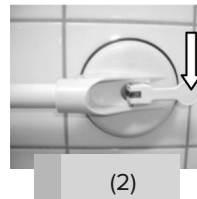
## Product Content

Suction grab bar with 2 suction cups. Suction grab bar is ready for use.

## Assembling

Assembling requires no tools. Check the vacuum suction discs and the surface it is to be attached to. Clean the surface thoroughly. Press the rubber discs firmly to the surface. The tilting vacuum levers must be loose at this point and the whole of each rubber disc must lie on the surface. (1)

Ensure that the rubber discs are completely on the tile area or other surfaces so that the discs do not cover any grouting lines or other joints. The discs should be positioned at least 1-1.5 cm from all joints. (see 1) First press down one tilting lever and then the second. The resistance of the negative pressure generated must be clearly felt. The greater the resistance felt on the vacuum lever, the better the suction power of the grab bar. If you use the bar regularly, you will get a feel for the vacuum strength and the suction power generated. Before each use, test the firm hold of the suction device. If the adhesion seems insufficient, release the rail and reattach again.


**If the required suction power hasn't been achieved, the rail should not be used !**

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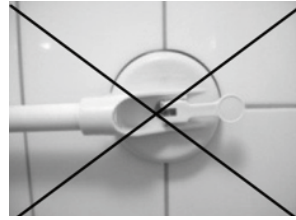
## Falsch / Wrong


Removal instructions: Release bath latches. Slide fingertip under rubber seal at each end to release suction.

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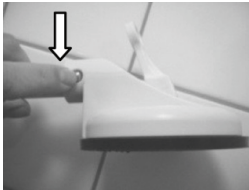## Suction cup exchange

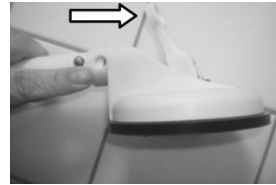For the exchange of the suction head press the interlock at the side and draw off the suction head in direction of the arrow. Push the new suction head until it lock clearly.

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## Safety Instruction

### ATTENTION:

This product is designed to provide support. It is not designed to hold a person's entire body weight. Before using the grab bar, clean and dry the area where the grab bar is to be attached, and read the instructions carefully.

**Be sure to test the stability of the grab bar before every use.**

Always make sure that the grab bar is securely attached before each use and always reattach if loose. **Do not use the suction grab bar if it is damaged!**

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## Cleaning / Maintenance

Always keep the rubber discs clear of oil and dust. Clean the rubber discs with standard household detergents (no acetic cleaner). Never expose the rubber discs to direct sunlight for longer periods otherwise they will quickly lose their elasticity. Check the rubber discs for possible deterioration. **If the suction cups show damage, replace immediately.**

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Stand: 10/2022

## Warranty

We issue a warranty of 24 months from date of purchase.

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EU Authorized Representative Drive Medical LTD
Drive Medical GmbH & Co. KG • Leutkircher Straße 44 • 88316 Isny/Allgäu • Germany