Bedienungsanleitung  
Instruction manual  
Manual de instrucciones  
Mode d'emploi

**TS-Aqua**


DE

EN

ES

FR

*Enjoy mobility.*

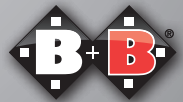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## Index

1.	Preface	25
1.1.	Signs and symbols	25
2.	Important safety instructions	26
2.1.	Indications	27
2.2.	Contraindications	27
2.3.	Service life	27
2.4.	Purpose	28
2.5.	Declaration of Conformity	28
2.6.	Responsibility	28
3.	Description of product and contents	29
3.1.	Inspection on delivery	29
3.2.	Contents	31
3.3.	Type plate and serial number	31
3.4.	Overview	32
4.	Assembly and adjustments	33
4.1.	The backrest	33
4.2.	The seat	34
4.3.	The arm rests	34
4.4.	The leg rests	34
4.5.	Wheels, tyres and Brakes	35
5.	Operation	37
5.1.	Getting in and out	37
5.2.	Moving around	38
5.3.	Using the commode	39
6.	Transport	39
7.	Storage and shipping	39
8.	Repairs	40
9.	Technical data	40
10.	Care instructions for the user	42
10.1.	Cleaning	42
10.2.	Disinfection	42
11.	Maintenance	43
12.	Disposal	43
13.	Transferral of the shower commode wheelchair	44
14.	Warranty	44

## 1. Preface

Dear user,

With the purchase of your Bischoff & Bischoff shower commode wheelchair in the TS-Aqua Series you are now the owner of a top-quality German product that sets new standards with its range of adjustments.

This instruction manual will familiarise you with the operation and use of your B+B shower commode wheelchair. Please read the instruction manual carefully before using the wheelchair for the first time.

### 1.1. Signs and symbols

Important! Indicates particularly useful information in the specific context.

Caution! Indicates information that is specifically relevant to safety. Follow the operating manual!

## 2. Important safety instructions

- Max. user weight 120 kg!
- Use the shower commode wheelchair only indoors and with the help of an assistant.
- Push it only on a flat and horizontal surface!
- When stationary, apply both the wheel brakes and also the parking brake.
- Avoid leaning too far out to the side. Risk of tipping!
- The shower commode wheelchair must be secured against use by unauthorised persons (especially children).

The use of a shower commode wheelchair is necessary if the user is unable to use or reach a toilet because they are disabled.

## 2.1. Indications

Walking disability or serious difficulty in walking as a result of:

- paralysis
- loss of limb
- defect or deformity in limbs
- joint contractures or joint injury
- other conditions

Shower commode wheelchairs are recommended if the patient cannot access standard commodes, showers or bathtubs without a wheelchair and the available space is free from barriers.

## 2.2. Contraindications

The shower commode wheelchair is unsuitable for persons:

- with reduced or restricted sitting ability
- with serious cognitive restrictions

## 2.3. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life of the shower commode wheelchair is up to 5 years. The shower commode wheelchair can continue to be used beyond this period, provided the product is in a safe condition.

## 2.4. Purpose

The shower commode wheelchair is designed exclusively to improve mobility or to transport handicapped persons for whom its use is indicated based on one of the above mentioned indications. It is designed for indoor use to compensate for mobility restrictions of the affected person when cleaning the body and going to the toilet. The wheelchair user can use the model with two large drive wheels without assistance and/or the model with four small steerable wheels with the aid of an assistant. The maximum user weight of 120 kg must not be exceeded.

Please note:

A warranty claim can only be accepted if the product is used under the specified conditions and for the specified purposes.

## 2.5. Declaration of Conformity

B+B as the manufacturer of the product declares that the TS-Aqua Variant 1 and 2 shower commode wheelchair fully complies with the requirements of Directive 93/42/EEC.

## 2.6. Responsibility

We are able to provide warranty cover only if: the product is used in accordance with the specified conditions and for the intended purposes, if modifications, extensions, repairs, and maintenance work are carried out only by persons authorised to do so by us and if the shower commode wheelchair is used in accordance with all instructions for use.

## 3. Description of product and contents

### 3.1. Inspection on delivery

All B+B products are inspected before they leave our factory and the CE mark is applied.

The B+B shower commode wheelchairs are delivered in a special box with a tool set. The wheelchair is delivered and instruction in its use is given by a qualified dealer in medical products.

If the wheelchair is delivered by rail or road transport it must be inspected for shipping damage in the presence of the delivery person.

If any damage is found, please note the following:

- record the facts
- prepare a declaration of assignment; all claims resulting from the damage are assigned to the delivery company
- return the record of facts, consignment note and declaration of assignment to Bischoff & Bischoff GmbH.

#### ***In the event of shipping damage...***

- Leave the goods and packaging undisturbed and do not use the goods.
- Report the damage to the transport company in writing as instructed below.
- Contact our customer service +49 (0)700/6000 7070.
- A representative of the transport company must be informed of any damage without delay in order to make a claim for compensation.

Externally obvious damage or losses must be confirmed in writing on consignment note by the delivery person before accepting the package. Concealed damage that is not obvious until the package is opened must be reported immediately in writing to the transport company.

All deadlines must be met:

- courier service and post within 24 hours
- rail delivery within 7 days
- road transport within 4 days after receipt of the package.

Bischoff & Bischoff GmbH will require the original consignment note with confirmation of damage.

Please pay particular attention to the highlighted sections.

Compensation from the insurer is not possible without these documents.


### 3.2. Contents

The scope of supply in the special box includes:

- Pre-assembled shower commode chair
- Commode container
- Instruction manual

### 3.3. Type plate and serial number

The type plate (see fig. 01) and serial number are located under the seat.

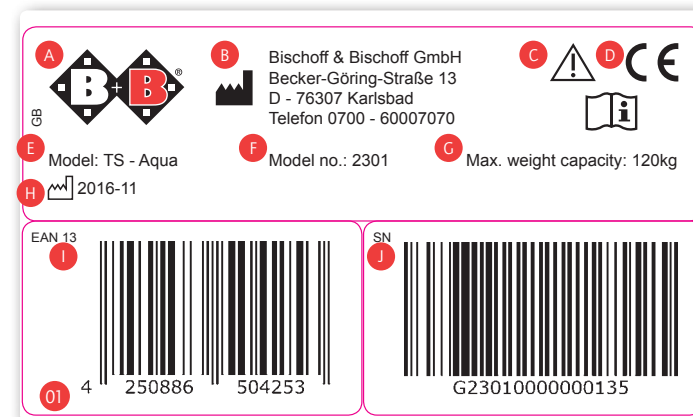- A Manufacturer logo
- B Manufacturer information
- C Important - Follow instructions for use
- D CE mark
- E Model designation
- F Model number
- G Maximum load capacity
- H Date of manufacture
- I EAN 13 item number
- J Serial number

### 3.4. Overview

The instructions for use of your B+B shower commode wheelchair include detailed illustrations to make it as easy as possible for you to familiarise yourself with its use and handling.

See the following illustrations for the names and interaction of the various components (see Fig. 1a and 1b).

### 4. Assembly and adjustments

Your B+B shower commode wheelchair is delivered ready-assembled in a B+B original box. Attached parts are supplied separately to prevent damage during transport.

You can carry out the adjustments of your commode chair yourself with a little manual dexterity. If you have any queries, please contact your specialized dealer.

#### 4.1. The backrest

The backrest can be removed completely.



- Unscrew (anticlockwise) the two star grip screws on the rear of the shower commode wheelchair. (Fig. 2)
- The backrest can then be pulled out of the guide rails that lie on both sides, or pushed in to assemble it. (Fig. 3)

#### 4.2. The seat


Installing and removing the seat

- The seat of your shower commode wheelchair can be removed and replaced.
- The seat is fixed to the frame of the shower commode wheelchair by two curved clamps.
- To remove the seat, hold the front and back of the shower commode wheelchair and pull the seat with a light jerk.

Replace the seat by placing it on the frame and press down on the front and back.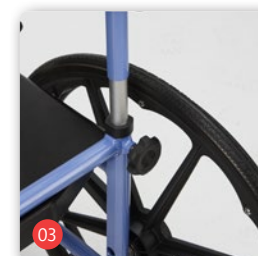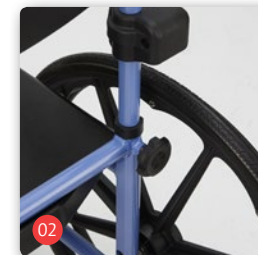

### 4.3. The arm rests

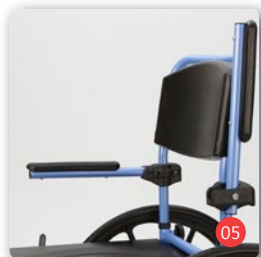
The arm rests of the TS-Aqua can be folded up.

**Attention!**  
Do not lift the TS-Aqua by the arm rests.

### 4.4. The leg rests

#### Folding up the leg rests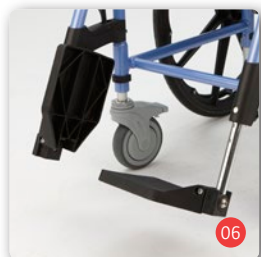


The leg rests can be folded upwards by simply lifting them. This can be useful when storing the TS-Aqua in a restricted space.

#### Leg rest height adjustment

The leg rests can be adjusted in height according to the individual length of the lower leg of the user.

- Undo the hand-nuts.
- Pull the bolts out.
- Slide the leg rest along the guide rail to the desired hole position.
- A suitable height of the leg rests is achieved if the user's thighs rest flat on the seat when their feet are resting on the leg rests.
- Insert the bolt into the desired hole position.
- Then tighten the hand-nut clockwise.
- Repeat the procedure on the opposite side.

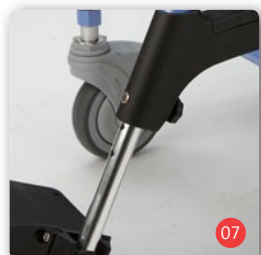### Folding away and removing the leg rests

To get in and out the commode wheelchair, the leg rests can be swung away and removed.

- Undo the locking mechanism (Fig. 08).
- Swing the leg rests outwards or inwards.
- To remove them, first fold the leg rests outwards; they can then be pulled up and out of their mountings.
- Fitting is the reverse of removal.

### 4.5. Wheels, tyres and Brakes

#### Wheels and tyres

Your TS-Aqua shower commode wheelchair is fitted with 5" wheels, all with brakes (variant 1), as well as with 24" drive wheels (variant 2). All are fitted with PU tyres as standard.

In the event of wear (worn tread, thread depth < 1 mm) or defect, repairs should be performed by the authorised dealer.

#### Wheels with brakes with directional locks (variant 1, Fig. 09)

Variant 1 of the TS Aqua is fitted with brakes on all of the steerable wheels. To activate the foot brake, step on the overhanging part of the pedal to push it down. To release the brake, step on the part of the pedal nearest to the wheel to push it down.

#### 24" drive wheels with quick-release axle (Variant 2)

The 24" wheels on your shower commode wheelchair are fitted with quick-release axles, which make transport of the wheelchair much easier. The drive wheels with the quick-release axles can be removed without tools.

### Installing and removing the drive wheels

- Press on the locking button of the quick-release axle with a finger and remove the wheel.
- Press the locking button of the quick-release axle again to replace the wheel.
- When installing the wheel make sure that the quick-release axle is correctly locked.

After installing the wheel make sure that it is secure by pulling it towards you without pressing the locking button of the quick-release axle.

### Setting the parking brake (toggle brake)

- To set the brake just press the brake lever forward until it locks (Fig. 11a)
- To release the parking brake, just pull the brake lever back (Fig. 11b)

### Adjusting the parking brake (Fig. 12)

The brake can be adjusted by moving the parking brake on the frame of the shower commode wheelchair. The compact brakes operate on only one drive wheel each.

If you find that the braking is inadequate or uneven, proceed as follows:

- Check the tyre tread depth. Check also that the friction is even. If the tread depth in the centre of the tyre tread is < 1 mm, the tyre must be replaced by the specialized dealer.
- To adjust the brake, undo the two screws.
- Now slide the brake in the frame guide to the point where the distance between the tread of the wheel and the brake bar is 16 mm.


Adjustments of the braking system should be made by the authorised dealer only!

### Tipping aid

A tipping aid can be fitted to Variant 2 which has hand rims.

- To do this, insert the tipping aid guard from below into the rear frame tube and tighten the securing bolts.
- For removal, the securing pins must first be pulled out; then the tipping aid guard can be pulled out.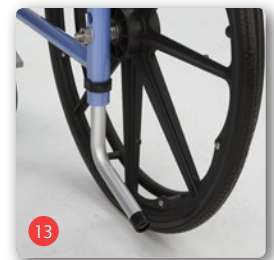

## 5. Operation

### 5.1. Getting in and out

It is easy to get in and out of the TS-Aqua. The foot plate and the two arm rests can be folded out of the way.

#### Getting in and out from the side:

- Move the shower commode wheelchair as close as possible to the seat on which you are sitting.
- Make sure that the seat is in a secure and stable position.
- Lock the shower commode wheelchair by setting the parking brakes.
- Fold the arm rest on your side of the wheelchair up out of the way.
- Then fold the foot plate up.
- Now slide sideways onto the seat of the TS-Aqua.
- Make sure that you are sitting as far back in the seat as possible.
- Now the arm rest and the foot plate can be returned to their normal position.

**Getting in and out from the front:**

- Move the shower commode wheelchair as close as possible to the front of the seat.
- Lock the shower commode wheelchair by setting the parking brake.
- Fold the foot plate up.
- Make sure that the seat is in a secure and stable position.
- Now slide onto the seat of the TS-Aqua by turning your body around. Make sure that you are sitting as far back in the seat as possible. Now fold the foot plate down and place your feet on it.

**5.2. Moving around**

Variant 1 of the shower commode wheelchair is moved around by an assistant. The assistant can move, steer and brake the shower commode wheelchair with the handles.

Variant 2 of the shower commode wheelchair can also be moved with an assistant, and in addition the user can move it independently with the drive wheels. Practise moving it carefully to start with until you are used to moving around with the shower commode wheelchair.

Shower commode wheelchairs have only limited anti-tip and anti-slip security. Particular care is necessary when braking, starting to move or turning on sloping ground. Leaning out of the shower commode wheelchair to an extreme degree increases the risk of tipping over.

**5.3. Using the commode**

**Using the commode**

Remove the commode container from the shower commode wheelchair by pulling it out of the guides from the back.

**Usage on the toilet**

The shower commode wheelchair can be used on the toilet without the commode container. First remove the commode container as described above. Now the TS-Aqua can be moved over any standard toilet.

**Lifting points**

You can lift the commode wheelchair by the metal frame in order to carry it over obstacles or stairs. Avoid lifting the commode chair by the seat and backrest.

**6. Transport**

To transport the wheelchair as easily as possible all removable parts (drive wheels and seat back) must be removed and the foot plate must be folded up.

**7. Storage and shipping**

If the wheelchair is to be stored or shipped, all removable and unsecured parts must be removed and packed separately in boxes. The separately packed parts can then be packed into one larger box. We recommend keeping the original packaging so it is available for this purpose if required. It provides optimum protection for your wheelchair from environmental influences during storage or transport.

## 8. Repairs

Repairs and maintenance must be done by authorised dealers only (medical suppliers or specialist dealers). To find a dealer in your area please contact our customer service department at +49 (0)700/6000 7070.

## 9. Technical data

Product designation:	TS-Aqua
Model:	Variant 1 with braked wheels Variant 2 with hand rims
Product number:	2300000000 (var. 1) 2310000000 (var. 2)

### Dimensions

Total length with leg rests (mm):	925 (var. 1) 1100 (var. 2)
Width (mm):	560 (var. 1) 630 (var. 2)
Length without leg rests (mm):	600 (var. 1) 775 (var. 2)
Height (mm):	990 - 1040
Seat angle (°):	0
Effective seat depth (mm):	430
Seat width (mm):	460
Seat height (mm):	505 - 555 (in increments of 25)
Backrest height (mm):	480
Lower leg length (mm):	410 - 535 (in increments of 25)
Height of armrest (mm):	210
Length of armrest (mm): (from backrest)	330
Length of armrest (mm):	245
Width of armrest pad (mm):	45

### Weights

Weight of the user (kg):	120
Empty weight (kg):	14.40 (var. 1) 19.20 (var. 2) (incl. pan)

### Tyres

Diameter of the hand rims (mm):	520
Rear tyres:	5" x 30 mm wheels with brakes 24"x 1 1/8" PU tread and plastic rim
Front tyres:	5" x 30 mm wheels with brakes

### Materials

Frame:	Lacquered aluminium
Attachments:	plastic (PU and PA fibreglass-reinforced)
Seat cushion:	padded PU, 430x460x40 (with hygiene cut-out)
Back cushion:	padded PU 230x460x30
Armrest cushion:	padded PU 245x45x20

### Ambient conditions

Temperature:	0 to 50 °C
Air humidity:	20 to 100 %

### Storage conditions

Temperature:	-20 to + 60 °C
Air humidity:	45 to 60 %

## 10. Care instructions for the user

### 10.1. Cleaning

Clean the material of the seats and seat backs with warm water and a sponge and/or a soft brush. A standard cleaning agent can also be used for stubborn soiling.

Note: never use aggressive cleaning agents such as solvents, scouring agents or hard brushes.

Plastic parts are best cleaned with a standard plastic cleaner. Please pay particular attention to the product information of the cleaning agent.

### 10.2. Disinfection

The shower commode wheelchair can be disinfected using a general-purpose household disinfectant. Use approved spray-disinfectants or alcohol-based wipe-down disinfectants.

A list of the disinfectants approved by the Robert Koch Institute can be found at [www.rki.de](http://www.rki.de)

## 11. Maintenance

Your shower commode wheelchair is made of high-quality materials for a long life of comfortable use. Every product is thoroughly checked before it leaves our premises. Nevertheless it is possible a defect may arise. If a defect arises with your shower commode wheelchair, do not continue to use it. With regard to repairs, contact the specialized dealer who supplied the product to you. He will be pleased to assist you. We further recommend that the shower commode wheelchair is serviced at least once a year. Your specialist dealer receives a servicing schedule from us, together with the documentation required for inspections. Our Customer Service department will be pleased to answer any questions you may have with regard to process, and to offer advice and information on the free-phone number 0700/6000 7070.

You must have your shower commode wheelchair regularly checked in accordance with the service schedule, and verified as being in good condition.

## 12. Disposal

If your shower commode wheelchair is no longer being used and is to be disposed of, please contact your specialist retailer.

If you wish to dispose of the product yourself, contact local recycling companies to enquire about the regulations that apply to your place of residence.

### 13. Transferral of the shower commode wheelchair

The shower commode wheelchair is suitable for re-use. Before it is transferred to another user it should be serviced by the specialist dealer and hygienically cleaned. When transferring the shower commode wheelchair to another user, please remember to give the new user this operating manual and the address of your specialist retailer.

### 14. Warranty

The warranty covers all defects of the shower commode wheelchair that can be traced back to a material or manufacturing defect. If you make a claim under the warranty, please submit the completed warranty certificate with the claim.

We provide the statutory warranty of 2 years for this product.

Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your specialist retailer will be happy to provide you with these.

Any disregard of the operating manual, incorrectly performed maintenance work and/or technical modifications, and any additions (attachments) done without the approval of Bischoff & Bischoff GmbH shall result in the cancellation of the both the warranty and any general product liability.

**Warranty certificate**

**Product:**  
**TS-Aqua shower commode wheelchair variant 1 + 2**

Serial number: \*  
 \*(to be entered by dealer)

**Dealer:**

Date and stamp  
 (Please note our general business terms and conditions)

\*See the side frame for information stickers

**Tip!**

In spite of careful research and editing, incorrect information may have crept into our operating manuals. The current version of each of the operating manuals can be found in the download area of our website: [www.bischoff-bischoff.com](http://www.bischoff-bischoff.com)

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### **B+B France S.A.R.L.**